

HUGE

GumEasy™

A-Silicone for Gingival Mask

Duplication Silicone Material

A-Silicone for Gingival Mask is the addition-curing silicone recommended for reproducing gingival morphology in implant as well as crown and bridge restoration.



Your Comprehensive Digital Dentistry Partner

Visualize, Optimize, Digitize

GumEasy™ Hard & Soft (CAD)

The scannable GumEasy™ is your most reliable partner for digitalization.

- ✓ Precise and detailed periodontal reproduction
- ✓ Scannable without scan spray
- ✓ Excellent grinding performance
- ✓ Nice mechanical property: **Tear strength: 10 KN/m**

GumEasy™ Soft H

The highly flowable GumEasy™ Soft H is a standard setter in aesthetics, fluidity and accuracy in gingiva reproduction.

- ✓ Excellent fluidity, fill in tiniest gaps
- ✓ Precise and detailed periodical reproduction
- ✓ Effortless injection
- ✓ Good grinding performance

GumEasy™ Soft L

With its controlled low fluidity, GumEasy Soft L is a highly efficient and reliable solution for fabricating artificial gingiva.

- ✓ High viscosity, easier to control
- ✓ Accurate gingiva reproduction
- ✓ Effortless injection and operation
- ✓ Good grinding performance



Why is GumEasy™ irreplaceable in making gingival mask?

*Printing Resin VS GumEasy™

	3D Printing	★ ★ ★ GumEasy™
Tensile Strength	3.7 MPa	6.5 MPa
Tear Strength	11 kN/m	17 kN/m
Shrinkage Rate	High	Extremely Low

* Test data based on official test report and dental resin material. The properties can vary with part geometry, print orientation, print settings, and temperature.

*Light Body VS GumEasy™

	Light Body	★ ★ ★ GumEasy™
Tensile Strength	3.7 MPa	6.5 MPa
Hardness/Shore A	54	40 & 70
Tear Strength	11 kN/m	17 kN/m
Fluidity	Flow with pressure	Flow without pressure

* Test data from HUGE professional laboratory.



Recommended Application Scenarios		
Product type	Recommended in the presence of undercuts	Recommended in the presence of implants
A-Silicone for Gingival Mask (Hard Type)	★ ★ ☆	★ ★ ★
A-Silicone for Gingival Mask (Soft Type)	★ ★ ★	★ ★ ★

Technical features ▼


Product	Mixing ratio (Base: Catalyst)	Working time* (min:s)	Setting time* (min:s)	Hardness (Shore A)	Scannable without sprays	Workable with a bur
GumEasy HARD	1:1	1:30	10:00	70	✓	✓
GumEasy SOFT	1:1	1:30	10:00	40		✓
GumEasy SOFT (CAD)	1:1	1:30	10:00	50		✓

A Versatile Master

HUGE GumEasy™ is compatible with both direct and indirect techniques in gingiva periodical reproduction, suitable for implant as well as crown and bridge restoration.

Direct Technique

Implant Restoration



<1> Analogue impression

<2> Plasticine barriers

<3> Application of silicone separator

<4> Application of A-Silicone for Gingival Mask

<5> Creation of orientation fence

<6> Gypsum casting

<7> Final result

Indirect Technique

Crown & Bridge Restoration



<1> Master model

<2> Application of A-Silicone for Laboratory

<3> Mask removal

<4> Remove suitable gypsum gingiva

<5> Creation of venting channels for silicone injection

<6> Application of the separator to the mask

<7> Mask repositioning onto the master model

<8> Injection of A-Silicone for Gingival Mask inside the mask

<9> Gingival Mask injection complete (material oozes out of the venting channels)


<10> Mask removal

<11> Final result

Your workflow with GumEasy™ A-Silicone for Gingival Mask

Impression Taking

01 PERFIT Impression Silicone & Light Curing Tray

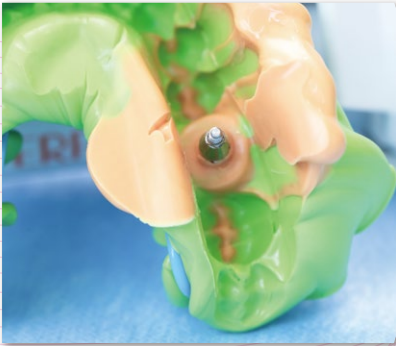

Model Fabrication

02 A-Silicone for Gingival Mask



Temporary Restoration

03 HUGE Multilayer PMMA Block



Bonding System

05 HUGE Dental Restorative Materials



Final Restoration

04 HUGE Zirconia Block



Packaging



Standard
2×50ml cartridges



Gingiva Elite Kit
2×50ml cartridges +12 mixing tips+12 intraoral tips+1*10ml Dental Separator for Silicones

HUGE

HOW TO CONNECT?

HUGE DENTAL Worldwide

Email: marketing@hugedental.com

Web: www.hugedental.com

United States

Tel: +1 626-283-5808

Email: info@hugedentalusa.com

Web: www.hugedentalusa.com



Shandong Huge Dental Material Corporation

Add / No.68 Shanhai Road, Donggang District, Rizhao City,
Shandong Province, 276800, P.R. China. Tel /+86 (633) 2277268
marketing@hugedental.com www.hugedental.com

Rev 06/2026



Follow @Huge Dental